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14501
Call button with red reassurance light - 2 modules.

CHARACTERISTICS.

- Supply voltage: 24 V d.c. ± 20% SELV
- Absorption: 30 mA max

OPERATION.

The device performs the function of calling nursing personnel and patient reassurance by means of a specific pilot lamp.

CONNECTIONS.

It is connected by traditional wiring to the display module 02081 or to the communication terminal 02080 for the room which acts as an interface with the bus line and the landing light (see the diagram "Connections").

14502
Call button for Call-way system, with 7-pin DIN socket for connection with tail call lead, red reassurance light - 2 modules.

CHARACTERISTICS.

- Supply voltage: 24 V d.c. ± 20% SELV
- Absorption: 30 mA max

OPERATION.

The device performs the function of calling nursing personnel and patient reassurance by means of a specific pilot lamp. It is equipped with a 7-pin DIN socket for the tail call lead and it is usually installed near to a headboard.

CONNECTIONS.

It is connected by traditional wiring to the display module 02081 or to the communication terminal 02080 for the room which acts as an interface with the bus line and the landing light (see the diagram "Connections").

14503
Cord-operated call button with red reassurance light - 2 modules.

CHARACTERISTICS.

- Supply voltage: 24 V d.c. ± 20% SELV
- Absorption: 30 mA max

OPERATION.

The device is equipped with 1.5 m of cord together with a red disc and it is usually installed in bathrooms and showers.

CONNECTIONS.

It is connected by traditional wiring to the display module 02081 or to the communication terminal 02080 for the room which acts as an interface with the bus line and the landing light (see the diagram "Connections").

14504
Cancel button with acoustic signal for receiving other calls and green indicator light - 2 modules.

CHARACTERISTICS.

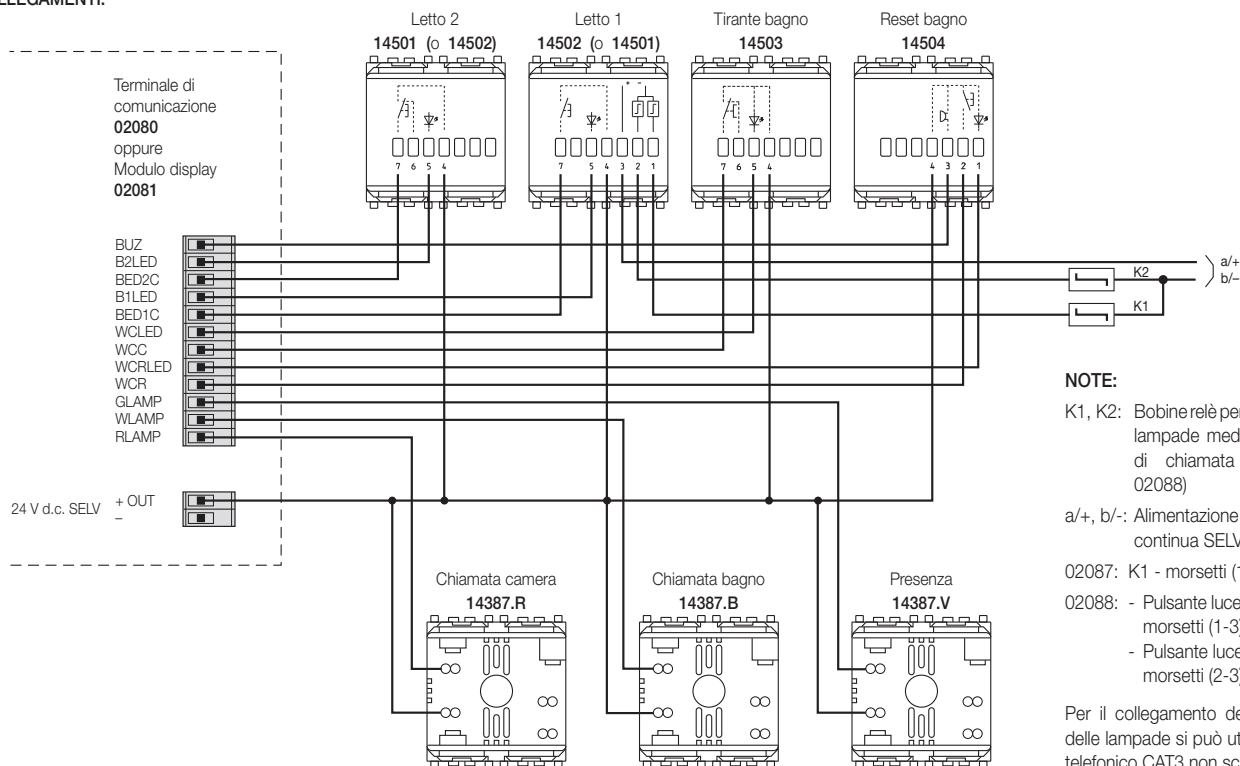
- Supply voltage: 24 V d.c. ± 20% SELV
- Absorption: 30 mA max

OPERATION.

The device performs the function of cancelling the call and signalling the presence of personal to the system; it is equipped with a buzzer for the acoustic signal if calls come from other rooms when the health personnel are not in the control room (calls are forwarded to the room where the personnel are providing assistance).

CONNECTIONS.

It is connected by traditional wiring to the display module 02081 or to the communication terminal 02080 for the room which acts as an interface with the bus line and the landing light (see the diagram "Connections").

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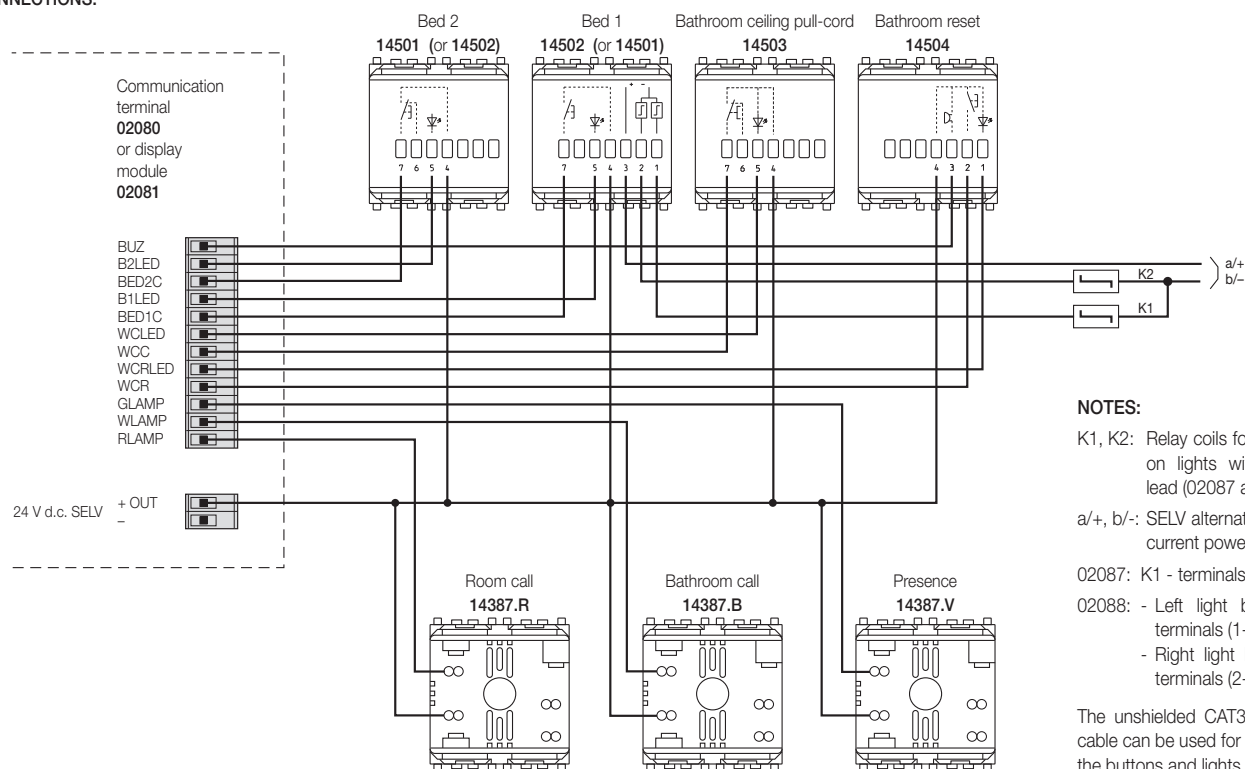
CONNECTIONS.

NOTES:

K1, K2: Relay coils for switching on lights with tail call lead (02087 and 02088)

a/+, b/-: SELV alternating / direct current power supply

02087: K1 - terminals (1-3)

02088: - Left light button: K1 terminals (1-3)

- Right light button: K2 terminals (2-3)

The unshielded CAT3 telephone cable can be used for connecting the buttons and lights.

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WEEE - Information for users

If the crossed-out bin symbol appears on the equipment or packaging, this means the product must not be included with other general waste at the end of its working life. The user must take the worn product to a sorted waste center, or return it to the retailer when purchasing a new one. Products for disposal can be consigned free of charge (without any new purchase obligation) to retailers with a sales area of at least 400 m², if they measure less than 25 cm. An efficient sorted waste collection for the environmentally friendly disposal of the used device, or its subsequent recycling, helps avoid the potential negative effects on the environment and people's health, and encourages the re-use and/or recycling of the construction materials.